# POSITIVE - NEGATIVE POLYVALENT CONTROLS

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## BIBLIOGRAFIA

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# POSITIVE - NEGATIVE POLYVALENT CONTROLS

Positive and Negative Polyvalent Controls to be used in Bacterial Serology kits with Micro, Macro and Slide Method

4 x 1 ml

REF 6500

## CHARACTERISTICS

Positive control is a human based serum with added anti-Salmonella Typhi and Paratyphi (A-B-C), anti-Brucella and anti-Proteus (OX 2-OX 19-OX K) antibodies. The title is such to guarantee a clear agglutination in the three methods and for every analyte.

Negative control is a non-reactive serum to Bacterial Serology analytes.

## REAGENTS

Kit composition:

REF 6500

**REAGENT 2** (liquid, red cap) 2 x 1 ml

Positive Control, ready to use.

**REAGENT 3** (liquid, blue cap) 2 x 1 ml

Negative Control, ready to use.

**STABILITY**: sealed and stored at 2-8°C, the product is stable up to expiry date printed on the package.

**NOTE**: human serum was used to prepare the product.

The raw material comes from apparently healthy donors, resulted negative to the administer for the research of anti-HIV/HTLV III and HBsAG antibodies.

Anyway, as it is not possible to exclude with absolute certainty HIV/HTLV III and HBsAG infection dangers, it is advisable to handle the product with the precautions required for human blood and its derivates.

## PREPARATION OF THE CONTROLS

Controls are ready to use, except dilutions related to the used procedure.

## BIBLIOGRAPHY

Available upon request.

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Issue 02 - Mar 2015 MS

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Via Fermi, 12 - 37026 Pescantina - VERONA - ITALY

Tel. +39 045 6700870 - Fax +39 045 7157763

sito web: <http://www.fardiag.com> e-mail: [fardiag@fardiag.com](mailto:fardiag@fardiag.com)

Manufactured by: **FAR** srl

Via Fermi, 12 - 37026 Pescantina - VERONA - ITALY

Phone +39 045 6700870 - Fax +39 045 7157763

website: <http://www.fardiag.com> e-mail: [fardiag@fardiag.com](mailto:fardiag@fardiag.com)