

ALBUMIN STANDARD

STANDARD ACQUOSO
PER IL DOSAGGIO DELL'ALBUMINA NEL SIERO

1 x 5 ml

REF 7514

PRINCIPIO

In soluzione tamponata a pH 3.8 l'albumina presente nel campione reagisce con il verde bromocresolo (BCG) provocando un cambiamento di colore, la cui intensità è proporzionale alla concentrazione di albumina presente nel siero o nel plasma.

REAGENTI

I reagenti sono standard acquosi pronti all'uso.

Attenzione: non inquinare le soluzioni; usare sempre puntali monouso.

| | | |
|---------------------------|-----------------|-----------------|
| Composizione del kit: | REF 7514 | Quantità |
| STANDARD (liquido) | 7514S | 1 x 5 ml |

ATTENZIONE: chiudere immediatamente i flaconi dopo l'uso.

STABILITÀ: a 2-8°C si conserva inalterato fino alla data riportata sulla confezione.

CONCENTRAZIONE DELLO STANDARD

Il calibratore ha la seguente concentrazione:

| | |
|-------------------|--------|
| Standard Albumina | 4 g/dl |
|-------------------|--------|

PROCEDIMENTO

Utilizzare il kit ALBUMIN STANDARD per la calibrazione durante l'impiego del kit ALBUMIN liquid (**REF** 7008).

Seguire la procedura indicata nella metodica ALBUMIN liquid.

REFERENCE

1. Dumas B. T., Watson W.A., Biggs H.g., clin. Chim. Acta 31, 87 (1971)



Edizione 02 - mar 2015 RR



Prodotto da:

FAR S.r.l.

Via Enrico Fermi, 12

37026 Pescantina VR

Tel. 39 045 6700870 - Fax +39 045 7157763

Sito web <http://www.fardiag.com> e-mail: fardiag@fardiag.com

ALBUMIN STANDARD

AQUEOUS STANDARD
FOR ALBUMIN DETERMINATION

1 x 5 ml

REF 7514

PRINCIPLE

In a pH 3.8 buffered solution the albumin in the sample reacts with bromocresol green (BCG) and causes a color change. The color intensity is proportional to the albumin concentration in the serum or plasma.

REAGENTS

Reagent is ready to use.

WARNING: do not pollute reagent. Use only disposable tips.

| | | |
|--------------------------|-----------------|-----------------|
| Kit composition: | REF 7514 | Quantity |
| STANDARD (liquid) | 7514S | 1 x 5 ml |

WARNING: close the vials immediately after use.

STABILITY: stored at 2-8°C, reagents are stable up to the expiration date.

STANDARD CONCENTRATION

| | |
|------------------|--------|
| Albumin Standard | 4 g/dl |
|------------------|--------|

ASSAY PROCEDURE

Use ALBUMIN STANDARD kit for calibration of ALBUMIN liquid (**REF** 7008).

See insert procedure for ALBUMIN liquid.

REFERENCE

1. Dumas B. T., Watson W.A., Biggs H.g., clin. Chim. Acta 31, 87 (1971)



Issue 02 - mar 2015 RR



Manufactured by:

FAR srl

Via Fermi, 12

37026 Pescantina - VERONA - ITALY

tel. +39 045 6700870/6700871 - fax +39 045 7157763

website: <http://www.fardiag.com> e-mail: fardiag@fardiag.com