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(*) Dangerous reagents are marked by an asterisk.
Refer to safety data sheets.

CPT Iyo

ANTICOAGULATIVE ANTIAGGREGANT
TO BE USED IN CASE OF SUSPECTED
PLATELET AGGREGATION

For in vitro diagnostic use only

Packaging:

100 x 3 ml

Code EM01-100

PRINCIPLE

CPT is a complex with anti-coagulative and anti-aggregant properties. Piridoxal phosphate prevents platelet aggregation caused by EDTA or by other anti-coagulative, citrate included.

It helps to prevent frequent laboratory adulterations: pseudotrombocytopenia seldom followed by pseudoleukocytosis.

If not detected, pseudotrombocytopenia may cause wrong diagnostic results and useless or unsuitable therapeutic treatments.

Moreover, unlike what happens with other anti-coagulative, CPT maintains stable the average platelet volume (MPV), the platelet volume distribution range (PDW) and piastrinocrite (PCT), allowing the correct determination with "Coulter" systems.

CPT helps platelets to maintain the same properties they have in whole blood, without anticoagulative after 24 hours. Also, all other hematological parameters remain the same.

REAGENT 1 (Lyophilized)

100 tubes

Citrate buffer 31 mmol/L
Sodium phosphate
Piridoxal-5'-phosphate 33 mM

STABILITY: up to expiry date on the label if kept away from light and stored at 2-8°C.

SAMPLE

Fresh whole blood.

PROCEDURE

The vacuum tube is filled with 3 ml of blood using the Vacutainer collection method.

In case the collection with syringes, inject 3 ml of blood using the perforable cap or removing the cap.

Once filled, shake the tube by hand or by vortex for about one minute.

Assay the hematological parameter within 24 hours from collection.

Store sample at 2-8°C.

WARNING

Using systems different from "Coulter" it has been noted an increase in volume for red corpuscles with consequent increase of hematocrite.

It is recommended to use the product only in case of suspect "cumpling" (platelet aggregation suspect) for a correct evaluation of piastrinemies.

NOTE

The appearance of the lyophilized tablet (i.e. a collapsed tablet) does not affect the test results.

KEY SYMBOLS

	In Vitro diagnostic medical device
	batch number
	catalogue number
	temperature limits
	use by
	caution
	read instructions for use

IVD

Ed. 05 - Mag 2023 RR/MS

MANUFACTURER

Via Fermi, 12 - 37026 Pescantina - VERONA - ITALY
phone +39 045 6700870
website <http://www.farddiag.com>
e-mail: order@farddiag.com
e-mail: farddiag@farddiag.com